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D - KP

Instructions for Pessary Treatment

Dr. Arabin GmbH & Co KG im FEZ
Alfred-Herrhausen-Str. 44, 58455 Witten
Tel: (49) 0 2302 189214 Fax: (49) 0 2302 189216
www.dr-arabin.de info@dr-arabin.de

Instructies

Dr. Arabin GmbH & Co KG im FEZ
Alfred-Herrhausen-Str. 44, 58455 Witten
Tel: (49) 0 2302 189214 Fax: (49) 0 2302 189216
www.dr-arabin.de info@dr-arabin.de

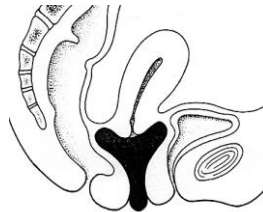
Club Pessary

Application: The club pessary consists of flexible silicone. The stem of the club pessary is supposed to avoid the gliding of the upper lying basis. **Only those patients with grade 3 to 4 prolapse of vagina and uterus are treated with the club pessary, in cases where urethra, ring or bowl pessaries do not sustain the pressure and where a cube pessary might increase stress incontinence.** Grade 4 prolapse may represent a big problem for the patient and a challenge for the physician. Before inserting the pessary, the prolapsed organs have to be reposed. Thereby the contents of the hernia have to be examined. If other organs apart from the uterus are contained in the prolapse, it has to be carefully redisplaced with two hands.

Sizes: The club pessary is measured by the circumference of the bowl's diameter, the sizes range from minimum 50mm to maximum 90mm diameter. The pessary with the smallest diameter that just holds should be inserted.

Use: After inserting the pessary the patient and physician check whether the pessary holds. The patient should first cough or press in the recumbent position. By moving the stem it can also be controlled whether the pessary holds. The patient should be explained the function of the pessary and should feel the stem herself after the first insertion. **Insertion will be facilitated by covering the pessary with estrogen cream.** Since a relatively large amount of cream is needed, we recommend to use a with 0.1 mg estriol / 1 g cream, in rare cases with up to 0.5 mg estriol / 1 g cream. The estriol also ensures a better blood circulation in the tissue and better epithel formation.

If possible, the patient should remove the pessary herself, e.g. while pressing. Occasionally, this may be difficult for the patient, since the pessary has to be edged on the pelvic floor by protecting the urethra.

Side Effects/Complications: **Loss of the pessary may happen during bowel movement. The patient should be instructed to feel the top of the club when having a bowel movement and, if necessary, to fix it in the vagina while pressing.**

If the pessary stays for some time (weeks), problems may arise and in worst scenarios even shrinkage processes in the lower third of the vagina. In those cases, a smaller club pessary or another type of pessary should be inserted. Most of the time, one has to chose a cube pessary, which should be changed daily.

Contraindications for estriol creams should be considered (e.g. pregnancy, lactation period, estrogen-dependent tumours).

If the patient is not able to change the pessary by herself, it may be advisable to integrate a nurse or a member of the family into the procedures.

The product must not be used by other patients and be stored by room temperature. **The pessary may be cleaned under warm running water without using any disinfectant.** The material can be cleaned and disinfected preferably with aldehyd-containing solutions (Gigasept FF). The pessary can also be sterilized by steam (134°C /3,8 bar) or hot air (250°C).

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