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D - SD

IT – SD

## Instructions for Pessary Treatment

Dr. Arabin GmbH & Co KG im FEZ  
Alfred-Herrhausen-Str. 44, 58455 Witten  
Tel: (49) 0 2302 1892 14 Fax: (49) 0 2302 1892 16  
[www.dr-arabin.de](http://www.dr-arabin.de) [info@dr-arabin.de](mailto:info@dr-arabin.de)

## Instructies

Dr. Arabin GmbH & Co KG im FEZ  
Alfred-Herrhausen-Str. 44, 58455 Witten  
Tel: (49) 0 2302 1892 14 Fax: (49) 0 2302 1892 16  
[www.dr-arabin.de](http://www.dr-arabin.de) [info@dr-arabin.de](mailto:info@dr-arabin.de)

## Vaginal Expander

**Application:** The vaginal expanders consists of a flexible silicone. Like all silicone pessaries they are elastic and can thus be inserted without pain. **Vaginal expanders are indicated in patients when an expansion of the vagina is aimed at for different reasons such as malformations, obstructions due to scars after episiotomies or vaginal operations.**

**Sizes:** The following sizes are available :

<b>Extra Small:</b>	20 mm diameter/ 120 mm length
<b>Small:</b>	26 mm diameter/ 127 mm length
<b>Medium:</b>	30 mm diameter/ 112 mm length
<b>Long:</b>	37 mm diameter/ 137 mm length

**Use:** The vaginal expander is indicated and adapted by the physician. The insertion in a relaxed standing or recumbent position is facilitated by covering the pessary with a gliding compound or estrogen cream. Exclusively creams containing estriol should be used (short adhesion to the receptor). Since one will need a relatively large amount of cream (large surface) to reach the gliding effect, we recommend to use a low dispensed estriol containing cream with 0.1 mg estriol / 1g cream, in rare cases with 0.5 mg estriol / 1g cream. The estriol also ensures a better blood circulation of the tissue. In addition, one can also use an antibiotic cream with does not destroy the natural vaginal flora. This treatment can be continued in case of strong discharge (1-2x/week).

**In general, the patient is trained to insert the expander herself.** The way, frequency and duration of the treatment will be prescribed by the physician, according to the indication and the aim of the treatment. Usually the vaginal expander is inserted in the evening and left over night.

**Side Effects/Complications:** The vaginal expander should be used with care. Contraindications for estriol cream should be considered (e.g. pregnancy, lactation period, estrogen-depending tumors).

The product must not be used by other patients and be stored by room temperature. **The pessary may be cleaned under warm running water without using any disinfectant.** The material can be cleaned and disinfected preferably with aldehyd-containing solutions (Gigasept FF). The pessary can also be sterilized by steam (134°C /3,8 bar) or hot air (250°C).

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