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MED/CERT
ISO 9003/EN 46003

Adaption-Set Ring

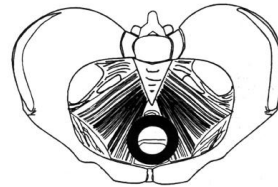
Application: The ring pessary set consists of 3 x 4 rings made out of flexible silicone. In contrast to the therapeutic ring pessaries the rings of the set do not contain the elastic spring steel and serve exclusively to adapt the right size of our pessaries such as ring pessaries, thick ring, hodge, sieve bowl, urethra-or urethra bowl or cube pessaries (see table below).

Sizes: Rings ordered in three separate sets a 4 rings ranging from 50mm to 100mm can be applied.

Small: 50 / 55 / 60 / 65 mm

Normal: 65 / 70 / 75 / 80 mm

Large: 85 / 90 / 95 / 100 mm

Use: The physician adapts the pessary on the first examination including tests like coughing, pressing and movements. After the test, the position of the pessary should be checked again and be removed.

The product must not be used by other patients and be stored by room temperature. **The pessary may be cleaned under warm running water without using any disinfectant.** The material can be cleaned and disinfected preferably with aldehyd-containing solutions (Gigasept FF). The pessary can also be sterilized by steam (134°C /3,8 bar) or hot air (250°C).

Advices for use of each pessary are applied within all packages.

Table to adapt appropriate sizes of a cube pessary on the basis of a test by a ring pessary

Ring Pessary	Cube Pessary
50 mm	Size 0/ up to 25 mm
55-60 mm	Size 1/ up to 29 mm
60-70mm	Size 2/ up to 32 mm
70-75mm	Size 3/ up to 37 mm
80-85mm	Size 4/ up to 41 mm
> 90mm	Size 5/ up to 45 mm

For further questions you may always contact our office.

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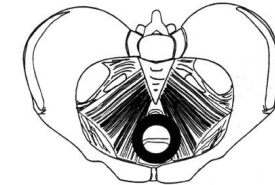
Ring Pessarium	Cubus Pessarium
50 mm	Size 0/ up to 25 mm
55-60 mm	Size 1/ up to 29 mm
60-70mm	Size 2/ up to 32 mm
70-75mm	Size 3/ up to 37 mm
80-85mm	Size 4/ up to 41 mm
> 90mm	Size 5/ up to 45 mm

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Dr. Arabin GmbH & Co KG im FEZ